


Facilitate™ Surgical Procedure


Preparation

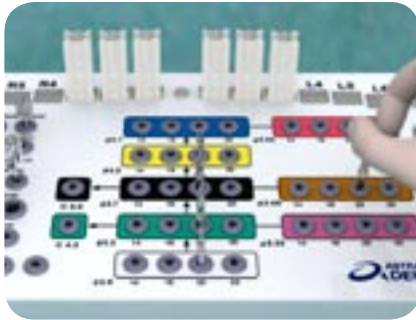

1. Follow the Drilling & installation protocol carefully and distribute the required implants and drills in their respective positions.



2. Mount the different specified Implant Holders in the implants and place back in their respective tray position.

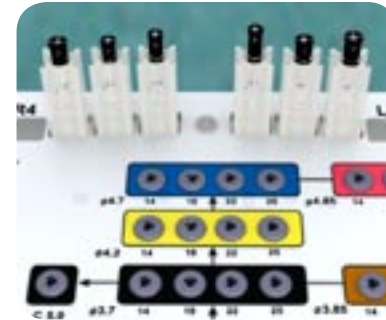

3. Make sure the surgical guide is in proper position, preferably with a bite registration index made in an articulator.

Punching



4. If applicable, punch or mark out the soft tissue punching positions. If needed, remove guide and complete incisions with punch, scalpel or other appropriate instrument.



5. Use the bit registration index and replace the surgical guide and fixate with screws.

Drilling



6. Follow the Drilling & installation protocol carefully, select the specified Drill Keys and drills. Place drill tip in Drill Key before starting to drill. Perform all drilling steps at a maximum speed of 1500 rpm under profuse irrigation. Drill until the stop sleeve touches the Drill Key.

Implant installation



7. Pick up the appropriate implant with the Implant Holder Driver mounted in the Contra Angle.

Facilitate™ Surgical Procedure

Implant installation



8. Perform the installation at 20 rpm under profuse irrigation. Recommended torque is 35 Ncm.



9. Continue until Implant Holder sleeve touches the surgical guide. Detach Contra Angle and leave Implant Holder in place until all implants are installed.



Guide Removal



10. When all implants are installed, loosen Implant Holder Screws. If needed, use Counter Holder to avoid rotation of the implant.



11. When all screws are loose, remove fixation screws and lift out the surgical guide.

Healing

Immediate loading



12. If any Implant Holder creates undercuts remove possible holders separately by using the fork end of a 3.7 mm Drill Key and lift with the holder screw.



13. If required, remove the screw completely and use the Implant Holder Remover. Screw in clockwise until holder comes out of the guide cylinder.



14. When the guide is removed install Healing Abutments or, if applicable, proceed with an immediate loading procedure.

